

VORTEX®

The Antistatic Chamber –
small, solid and effective



The inhalation aid with an aluminium chamber –
for treatment that works



Appealing masks for children:
„Mary the Ladybug“ and
„Felix the Frog“



Antistatic metal chamber
ensures reliably metered
dosing



The transparent mouthpiece
allows a visual check of the
medication spray

Adaptable for all age groups



VORTEX®
4 years and older
Item no. 051G1000 / 051G1001



VORTEX® with baby mask 'Ladybug'
0–2 years
Item no. 051G2020 / 051G2021



VORTEX® with child mask 'Frog'
2 years and older
Item no. 051G2040 / 051G2041



VORTEX® with adult mask
Item no. 051G1060 / 051G1061

Accessories



VORTEX® baby mask 'Ladybug'
0–2 years
Item no. 041G0712



VORTEX® child mask 'Frog'
2 years and older
Item no. 041G0713



VORTEX® mouthpiece
Item no. 051B0100



VORTEX® connecting ring
Item no. 051E5004



PARI GmbH Spezialisten für effektive Inhalation
Moosstraße 3 · 82319 Starnberg
Telefon +49 (0) 8151 279-0
Fax +49 (0) 8151 279-101

You can find PARI on the Internet:
E-Mail: info@pari.de · www.pari.com/de-en



Attach. Breathe.
Relax.

VORTEX®

Tips on handling
and hygiene



© PARI GmbH Spezialisten für effektive Inhalation 051D0035-B 06/2015



Specialists in effective inhalation



www.pari.com/de-en

Specialists in effective inhalation



Tips on handling

Preparing the metered dose inhaler

- 1 Remove the sealing cap on the metered dose inhaler (MDI). Shake the MDI vigorously.



- 2 Then insert the mouthpiece of the MDI into the blue connecting ring of the VORTEX®.



- 3 Remove the blue protective cap of the VORTEX®.



- 4 Hold the VORTEX® mouthpiece between your teeth and enclose it with your lips.

Press the MDI once with your thumb to trigger a puff of spray.



- 5 Breathe in slowly and deeply. Hold your breath briefly and then breathe out again through the mouthpiece. If the doctor treating you has prescribed two or more puffs per application, repeat steps 4 and 5.



Tips on hygiene

Disassembling the VORTEX®

- 1 Remove the connecting ring, and the mask if it is attached.

Leave the mouthpiece and the open protective cap on the inhalation chamber.



Cleaning

- 2 Clean all parts gently but thoroughly in warm tap water with a little dish washing liquid.



- 3 Rinse all parts thoroughly with fresh, clean water.



Children younger than 18 months should inhale 4 to 6 times, children older than 18 months should inhale deeply 2 or 4 times.



Quick reference guide.
Please read and follow the instructions for use.

Disinfection

- 4 The VORTEX® including the mask can be disinfected in a steam disinfectant.



- 5 Possible alternative: Place all the parts in boiling water for at least 5 minutes. Make sure that the VORTEX® is completely immersed throughout the process.



Drying

- 6 Allow the VORTEX® to dry at room temperature. Because of the antistatic properties of the VORTEX®, you can also dry it with a clean tea towel.



Storage

- 7 Fit all the parts of the VORTEX® back together again and store it safely in a dry, dust-free place.

